



# Lucent LED Operatory Light Quick Reference Installation Guide

## ASSEMBLING THE HEAD TO THE ARM

Remove adaptor  
Bushing

Attach bushing  
to light head



## ELECTRICAL CONNECTIONS


Connect wire  
harness

Secure light  
head to arm



strumentidenti.it

## Light Handles

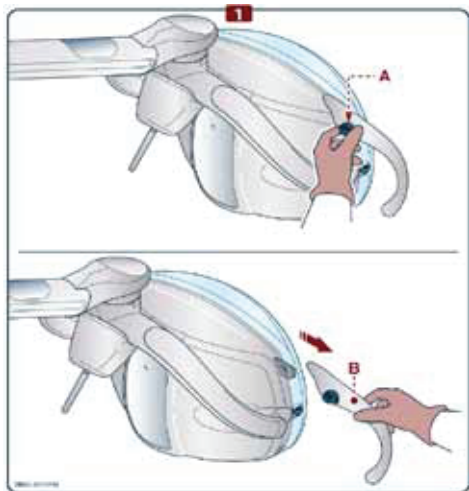

Switch on / off / Adjustment

- To switch the lamp On and Off, press and release the switch to the LEFT or RIGHT side.

When the lamp is switched On / Off, the device will beep (once).

DON'T push and pull the switch. It only moves in a left and right direction.

- Right & Left Light handle
- Light switch toggle extension
- Light shield wrench
- Light bushing pre installed into arm.

Signature \_\_\_\_\_ Date \_\_\_\_\_